# Procedures CONSULT

## proceduresconsult.jp

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The image shows two screenshots of the mobile application interface. The left screenshot shows the procedure overview for 'Abdominal Paracentesis' (腹腔穿刺) with tabs for 'Video', 'Procedure Summary', and 'Checklist'. The 'Checklist' tab is highlighted with an orange box and an arrow pointing to the right screenshot. The right screenshot shows the detailed checklist for 'Abdominal Paracentesis' with a 'Quick Review' and 'Checklist' tab. The checklist items are as follows:

- Explain the procedure and obtain consent.
- Secure monitoring line and an IV, if indicated.
- Observe universal precautions and sterile technique.
- Position the patient:
  - RLQ or LLQ approach: Supine with the head slightly elevated.
  - Midline infraumbilical approach: Left lateral decubitus.
- Assess landmarks:
  - RLQ or LLQ: 4 to 5 cm superior and medial to anterior superior iliac spine.
  - Midline infraumbilical: 2 cm below umbilicus.
  - Exclude massive hepatomegaly or splenomegaly.
  - Avoid scars or abdominal wall vessels.
- Prepare the skin at the puncture site (antiseptics), apply drapes, mark the site:
  - Midline: 2 to 3 cm below umbilicus (patient semi-upright or lateral)
  - Lateral: 3 to 5 cm medial and cephalad to anterior superior iliac spine (patient supine)
- Anesthetize entry site.
- Prepare the needle assembly.
- Direct the paracentesis needle perpendicularly through the skin, applying intermittent suction.
- Withdraw fluid for analysis.
- For therapeutic paracentesis, remove remaining fluid through tubing/Luer-Lok attached to evacuation containers.
- Remove the catheter, cover the site, apply direct pressure.
- Submit samples to the laboratory.

Copyright © 2010 Elsevier Japan. All rights reserved.  
[Terms and Conditions](#) | [Privacy Policy](#)

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